

PASS LP – DRC connector


1 Functionality

- The DRC (derotation) connectors are used to allow derotation maneuver of the spine.
- Use 2 above and 2 below apex curve for derotation of deformity in axial plane.
- The DRC connector must be used on both sides of the construct on the same vertebrae for derotation.
- Can be locked either on the rod or the screw head.
- One size fits both rod diameters 5,5 and 6,0mm.
- 2 version of nuts : breakable or not
- Do not constraint screw polyaxiality.

2 Specific instrumentation

- Specific DRC holder A02330300 must be used for the DRC connector orientation.
- T30 screwdriver A02230020 must be used to lock the connector setscrew
- Nutdriver A02130160 must be used to lock and break the breakaway nut

3 Illustration

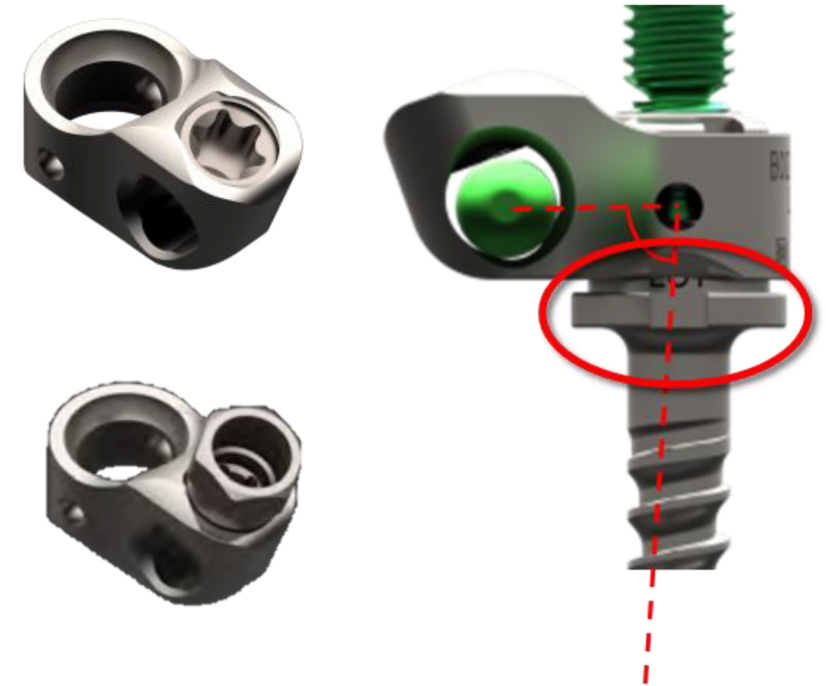

4 References

- B02236020 : without breakaway nut
- B02236023 : with breakaway nut